




ImmunoCard **STAT!** CRYPTO/GIARDIA

Direct Detection of *Cryptosporidium* and *Giardia lamblia*

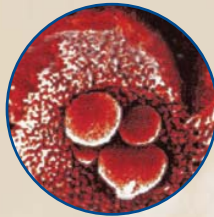

Can you choose between cost, speed and accuracy?

Using Meridian's ImmunoCard STAT!® *Crypto/Giardia* means not having to compromise at all. Now your lab can generate better results detecting the two most common causes of intestinal parasitic disease in less time and at less cost than traditional methods. Isn't that a simple choice to make?

“Stool immunoassays are very specific and more sensitive than the routine O&P examination... They are also less labor-intensive than the routine O&P examination and require less microscopy training and skill”¹

- Efficient, effective and economical detection of the two most common enteric parasites.
- Results in approximately 10 minutes.
- O&P Sensitivity: 75-80%²
ImmunoCard STAT!® *Crypto/Giardia* Sensitivity: 97-100%³
- Clearly defined test bands for rapid interpretation.
- One card, two results – both reimbursable.


Meridian
Bioscience, Inc.
Inspired Science. Trusted Solutions.

¹ Garcia, Lynne S; *Diagnostic Parasitology – Laboratory Information*. 10/2004

² Aziz H, Beck CE, Lux MF, Hudson, MJ. A Comparison of Different Methods Used in the Detection of *Giardia lamblia*. *Clinical Laboratory Science*. Summer 2001; 14, 3; 150-154.


³ Meridian Bioscience, Inc - ImmunoCard STAT!® *Crypto/Giardia* Package Insert

ImmunoCard STAT!® Crypto/Giardia Test Procedure

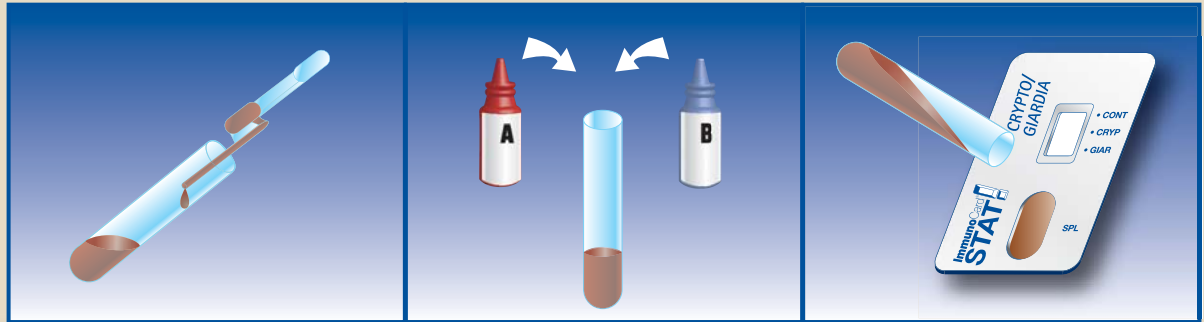

1. Add 2 drops of Sample Treatment Buffer to sample tube. Pipet 60 μ l of stool sample into sample tube.
2. Add 2 drops Conjugate A and 2 drops Conjugate B to sample tube and mix well.
3. Add tube contents to sample well of the device. Incubate for 10 minutes, visually interpret results.

Interpretation of Results

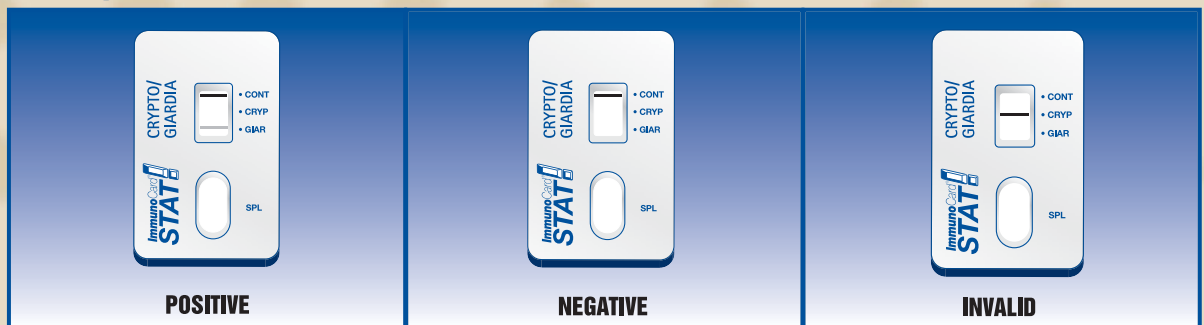

**This chart does not contain complete instructions for use. For further information, please refer to the package insert.*

Ordering Information

| Product | Test Size | Catalog # |
|----------------------------------|-----------|-----------|
| ImmunoCard STAT!® Crypto/Giardia | 30 Tests | 750830 |

Other Meridian Products

Meridian Specimen Transports

| | | |
|--------------|--------------------|--------|
| 10% Formalin | 120 vials per case | 900412 |
| C&S | 120 vials per case | 900612 |
| SAF Vials | 120 vials per case | 900212 |
| Clean Vials | 120 vials per case | 900312 |
| MF Vials | 120 vials per case | 900712 |

**Meridian
Bioscience, Inc.**
Inspired Science. Trusted Solutions.

USA/Corporate Office:
3471 River Hills Drive
Cincinnati, Ohio 45244
Telephone: 513.271.3700
Orders/Customer Service: 1.800.543.1980
Technical Support: 1.800.343.3858
Information Fax: 513.272.5432
Ordering Fax: 513.271.0124
www.meridianbioscience.com

Belgium / France / Italy / The Netherlands

©2006 ICSCG001

